# SUPER VIEW® Disposable Lens Set

REF 40411.000

Instructions for use

## Indications for Use:

Allows visualization of the ocular fundus, vitreous and retinal structures during vitreoretinal surgery.

## Intended Use:

Intended for use with the Insight Instruments SUPER VIEW® Wide Angle Viewing System or the BIOM®.

## Parts include:

- (1x) Wide Field Lens
- (1x) Correction Lens (for use with an f=175mm or f=200mm objective lens)
- (1x) Hassan-Tornambe Contact Lens
- (2x) Drive Belts (for use with BIOM®)
- (1x) Thumb Screw Cover (for dovetail)

## Warning:

Never use a product that appears to be damaged!

## Directions: Wide Angle Lenses

Non-sterile personnel

- Check the packaging for damage. The product must not be used if any damage is found on the packaging.
- After a visual check, open the outer pouch without touching the inner pouch.
- Let the inner pouch slide onto a sterile surface. Make sure the inner pouch is not contaminated during this process.

Sterile operating nurse/surgtech

- Open the inner pouch and let the contents slide onto a sterile surface.
- Remove the two lenses from the fleece packaging without touching the lens surfaces.
- Hold the correction lens (Figure 1-A) by its flange and place it in the lens holder (Figure 1-B) so that the mounting ring tabs on the lens fit into the notches in the lens holder.
- Turn the correction lens clockwise, passing through the resistance of the spring clip. The correction lens is now secure and will not fall out.
- Using your thumb and index finger, hold the wide field lens (Figure 1-D) on either side of the black arm without touching the lens surface (Figure 1). **Failure to hold the clip in this manner may result in the premature breaking of the clip.** The lens is correctly oriented if its position matches the diagram (Figure 2). Fasten the lens by inserting the black plastic tab into the slot (Figure 1-C) passing the resistance until the small outer tabs click and lock into position.
- To remove the lenses after use, grasp the wide field lens clip as shown (Figure 3) and simply break the lens clip in a firm downward motion. Remove the remaining portion of the clip and discard. Turn the correction lens counter-clockwise and carefully lift out of the lens ring holder.

## Directions: Hassan-Tornambe Contact Lens

- Swing wide angle module up and out of the microscope beam path.
- Flush the cornea thoroughly with ophthalmic solution.
- While taking care not to touch the lens surface, hold the lens only by its edge and place a "tear drop" amount of ophthalmic lubricant on the curved underside of the contact lens.
- Place on the cornea, curved side down and hold in place with a cotton swab for several seconds to displace any air bubbles and stabilize the lens on the eye.

## Directions: Thumb Screw Cover

- Grasp thumb screw cover by flat end and push over the dovetail thumb screw.
- When tightening or loosening thumb screw, grasp cover by the round portion.
- Remove cover after use by grasping flat end and pulling off of thumb screw.

## Directions: Drive Belts

- Please refer to BIOM® manufacturers instructions for use.

## Sterilization:

These items are delivered in a sterile condition and are for **SINGLE USE ONLY**. After surgery they should be properly disposed of according to contaminated medical waste guidelines of your facility.

## Contents sterile if package is not expired, opened, damaged or wet.

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