


# SUPER VIEW® WAM™ & Adapter Lubrication and Maintenance

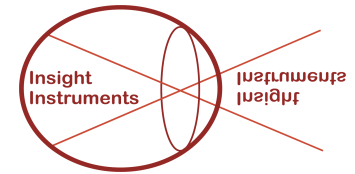

## Step 1:

Detach adapter from WAM™ housing

## Step 2:

Carefully place a drop of Surgical Instrument Oil\* in the following locations:

- (A) Teflon Washer
- (B) Teflon Washer
- (C) Ball Plunger
- (D) Adapter Pin
- (E) Swing Head
- (F) Lead Screw
- (G) Safety Rod, down the groove in safety rod
- (H) Receiver
- (I) Safety Rod Cap

## Step 3:

Rotate and move all moving parts and hinges

- Flip Swing Head hinge
- Engage Safety Rod several times
- Run the Elbow Tube from top to bottom on Lead Screw by turning the thumbwheel
- Spin adapter ring one full rotation

## Step 4:

Autoclave as approved (See the SUPER VIEW® Wide Angle Viewing System User Guide for Reprocessing for instructions and information on sterilization).

## NOTE:

After application of the lubricant, wipe any remaining lubricant with a clean lint free cloth.

\*Available from Insight Instruments

